

# Specially Designed Dental Elevators



Patented Luxating  
WingAngle Left / Right

**New & Improved**

GLux / WingLux

Luxating Elevator Titanium (*Sharpen Free*)

Removable Handle With Additional Tips

**IFU** *with*  
**Visuals**

If we don't have the instrument  
you're looking for,  
we will **MANUFACTURE** it for you!



[www.GerVetUSA.com](http://www.GerVetUSA.com)  
[sales@gervetusa.com](mailto:sales@gervetusa.com)

1188 Willis Ave Suite#801,  
Albertson, NY 11507

888 814 4650 - 516 385 3434 - 833 906 7575



# Glux Luxating Elevator Titanium Removable Handle with Additional Tips

**GerVetUSA™**  
Improving Instruments, Refining Surgeries

**New & Improved**  
**Glux™**  
(Sharpen Free)



GLT-2022-76 DT

**SALE: \$539.00**

**REG: \$659.70**

## GLUX LUXATING ELEVATOR TITANIUM REMOVABLE HANDLE WITH ADDITIONAL TIPS

GLux Luxating Elevator contains a thin spoon-shaped removable titanium tip. It's one of the most anticipated innovations in the surgical industry. The tips are made with titanium, making it adjustable, sharpen-free & long-lasting. To facilitate the technicians, we are also offering a free-of-cost extra tip. Different variations in these instruments assist veterinarians in getting the desired outcomes.

**1.5 mm**

GLT50-6950 DT

~~Reg: \$109.95~~  
**Sale: \$99.95**

**2 mm**

GLT50-6951 DT

~~Reg: \$109.95~~  
**Sale: \$99.95**

**3 mm**

GLT50-6952 DT

~~Reg: \$109.95~~  
**Sale: \$99.95**

**4 mm**

GLT50-6953 DT

~~Reg: \$109.95~~  
**Sale: \$99.95**

**5 mm**

GLT50-6954 DT

~~Reg: \$109.95~~  
**Sale: \$99.95**

**6 mm**

GLT50-6955 DT

~~Reg: \$109.95~~  
**Sale: \$99.95**

[www.GerVetUSA.com](http://www.GerVetUSA.com)



# Winglux Luxating

## Winged Elevator Titanium Removable

### Handle with Additional Tips



**New & Improved**  
**WingLux™**  
*(Sharpen Free)*



GLWT-2022-88 DT

**SALE: \$539.00**

**REG: \$659.70**

#### WINGLUX LUXATING WINGED ELEVATOR TITANIUM REMOVABLE HANDLE WITH ADDITIONAL TIPS

WingLux Luxating Winged Elevator is a cost-effective surgical instrument. It has a winged tip to reach at the narrow spaces feasibly. The best feature that makes it unique over others is its titanium variable tip. We are also offering a free-of-cost additional tip that will help you to keep the maintenance costs low.

**1.5 mm**

GLWT50-6940 DT

~~Reg: \$109.95~~  
**Sale: \$99.95**

**2 mm**

GLWT50-6941 DT

~~Reg: \$109.95~~  
**Sale: \$99.95**

**3 mm**

GLWT50-6942 DT

~~Reg: \$109.95~~  
**Sale: \$99.95**

**4 mm**

GLWT50-6943 DT

~~Reg: \$109.95~~  
**Sale: \$99.95**

**5 mm**

GLWT50-6944 DT

~~Reg: \$109.95~~  
**Sale: \$99.95**

**6 mm**

GLWT50-6945 DT

~~Reg: \$109.95~~  
**Sale: \$99.95**

# Patented Luxating WingAngle Left / Right Color Coated

*Left / Right*



**SALE: \$799.00**

**REG: ~~\$1,679.40~~**

## LUXATING WING ANGLE LEFT RIGHT STAINLESS STEEL COLOR COATED

Luxating WingAngle is a veterinary dental surgery instrument that makes tooth extraction feasible, reliable, and less time-consuming. It boasts a less concave blade to reach the periodontal ligament (PDL) root surface for enhancing it feasibly. We carry Silver, Rose Gold, Gun Metal, Gold, Blue and variable sizes. It is available in a set of 12 comprising 6 right angles and 6 left angles designed to meet the unique needs of surgeons.

✓ Periostome    ✓ Luxator    ✓ Crane pick    ✓ Chisel    ✓ Elevator    ✓ Proximotor

**1.5 mm**

GD50-8010 R  
GD50-8010 L

~~Reg: \$139.95~~  
**Sale: \$89.95**

**2 mm**

GD50-8011 R  
GD50-8011 L

~~Reg: \$139.95~~  
**Sale: \$89.95**

**3 mm**

GD50-8012 R  
GD50-8012 L

~~Reg: \$139.95~~  
**Sale: \$89.95**

**4 mm**

GD50-8013 R  
GD50-8013 L

~~Reg: \$139.95~~  
**Sale: \$89.95**

**5 mm**

GD50-8014 R  
GD50-8014 L

~~Reg: \$139.95~~  
**Sale: \$89.95**

**6 mm**

GD50-8015 R  
GD50-8015 L

~~Reg: \$139.95~~  
**Sale: \$89.95**



# Instructions for Use

## How to Place and Use WingAngle

**WingAngle™**

**1** Choose an appropriate tip size according to the size and position of the animal's tooth

**2** Hold the instrument with the handle in the palm and the tip of the index finger extended along the shaft

**3** Find the ligament space

**4** Place the angled edge of the WingAngle over the gingival margin

**5** Then, penetrate the angled edge of the blade into the PDL space

**6** Cut the periodontal ligaments (PDL) in several places by just sliding the angled edge

**7** From inside, move around the blade well and go as deep as you can.

**8** Switch to the other angle/side and repeat the above steps

**9** When the tooth gets loose enough, extract it with extraction forceps

**Important Note:**  
Don't push or rotate the Wingangle, as it is a delicate instrument used to cut the ligaments, not to elevate the teeth; otherwise, the tip of the instrument might break.

888 814 4650 - 516 385 3434  
833 906 7575

## How to Place and Use GLux Luxating Elevator Titanium Color Coated

**GLux™**

**1** Choose an appropriate tip size according to the size and position of the animal's tooth

**2** Hold the instrument with the handle in the palm and the tip of the index finger extended along the shaft

**3** Find the ligament space

**4** Place the tip of the GLux over the gingival margin

**5** Then, penetrate the tip into the PDL space

**6** Cut the periodontal ligaments (PDL) in several places by gently sliding and spinning the tip

**7** From inside, move around the blade well and go as deep as you can.

**8** When the tooth gets loose enough, extract it with extraction forceps

**Important Note:**  
Don't push or rotate the GLux, as it is a delicate instrument used to cut the ligaments, not to elevate the teeth; otherwise, the tip of the instrument might break.

888 814 4650 - 516 385 3434  
833 906 7575